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Beta (Glass) optical glass ionomer

Beta glass is a light-cured protective and filling tooth cavities substance, which has high compressive strength and strong adhesion to dentin and enamel.

This substance has fluoride-releasing property which can be used to infrastructure any kind of restoration.

The combination of light-cured resin and filler with Nano-sized particles has created excellent mechanical and chemical properties and this substance is very resistant to occlusal surface stresses.

Indications:

- Can be used as a liner and flooring under all types of filling materials
- Can be used as a fissure sealant
- Can be used for direct restoration of small defect

Properties:

- Easy to use and insert directly into restoration area
- Excellent chemical bonding to enamel and dentin.
- Contains fluoride to prevent secondary caries.
- High radiopacity.
- Easy handling.

Instructions:

To prepare the cavity: Grind the caries completely. Prepare the cavity with the minimum tooth reduction and with rounded interior lines. The cavity must be completely free from any kind of pollution and completely dry.

Protecting the pulp: No need for a liner, if there is no contact or proximity to the pulp, otherwise calcium hydroxide should be used to protect the pulp to prevent the infiltration of acrylic materials.

Etching: The next step includes etching. Apply the surface of the tooth with the etching gel and after 15 to 20 seconds, rinse the tooth cavity and dry the area completely.

Insert the material in a thin layer of about 1 mm in the cavity and cure, and if more thickness is needed, cure the glass in each step.

Hold the light source as close to the surface as possible (about 2 mm) and cure for at least 20 seconds.

The curing depth in 20 seconds is 1/3 mm and in 40 seconds is 3/5 mm. Complete the treatment operations using composite restorative materials according to the manufacturer's instructions.

Storage conditions:

- Close the syringe lid immediately after each use of glass ionomer.
- Keep at a temperature of 2 to 25 degrees centigrade.
- Keep away from direct sunlight and direct light sources.
- Do not use the product after the expiration date.
- If you keep the glass Ionomer in the refrigerator, you should keep it at normal room temperature for a while before using it.
- Keep out of the reach of children.
- The lifespan of the product is 3 years.

Precautions:

This product is made for use in dental restorations and should be used according to the manufacturer's instructions.

In case of contact with the oral mucosa, rinse with water and do not use it for people who are allergic to any of the components of this substance, and if they have any kind of allergic or skin sensitivities after using this substance, consult a specialist doctor.

Phenolic materials such as eugenol can cause lack of proper curing, so avoid using eugenol-containing cements as liner and in the vicinity of the tooth being restored.