BETA DENT

MAKU industrial park Free zone, Azerbaijan, IRAN
www.betadent.com +989102069694

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Acrosan Temp

Acrosan Temp is a self-curing acrylate polymer consisting of two components, powder and liquid, which is used to make temporary crowns and bridges to protect prepared teeth from external influences and maintain occlusion.

Advantages

- Match with the natural color of teeth
- Has 4 colors: bleach, A1, A2, A3 according to Vita standard
- Suitable setting and working time
- Compatible with other plastic crowns
- Suitable polish-ability and proper shining
- High strength
- Formability and easy to use

Working conditions with substance

- The mixing ratio of powder and liquid is 2 to 1.
- First, pour the liquid into the mixing container and then add the powder to the liquid and mix well for 30 seconds.
- After one minute, you can use the substance to take impression.
- Polymerization is completed within 10 minutes and after completing this stage, you can remove the excess and polish the crown-veneer.

How to use

First, take an impression of the restoration area and prepare a plaster cast and create the shape and form of the tooth on the cast.

Take an impression of the shaped and prepared cast and transfer the mixed powder and liquid materials to the silicone mold.

Place the silicone mold on the prepared area to make the crown or bridge and wait to complete the polymerization.

Remove excess material after complete polymerization and perform the final polishing and adjustment step.

Preferably use eugenol-free temporary cements to cement the crown-veneer.

The composite cure is considered with a light-cure device with a light intensity of 500 milliwatt per square centimeter and a length of 400 to 500 nanometers. The required time may vary depending on the intensity of the light and the depth of the restoration.

Maintenance Instructions

- Close the containers immediately after each use.
- Store at 10 to 25 degrees Centigrade.
- Keep away from direct sunlight and direct light sources.
- Do not use the product after the expiration date.
- If you store the product in the refrigerator, you should leave it at room temperature for a while before use so that the setting time can be carried out according to the company's instructions.
- Shelf life of the product is 3 years.

Safety recommendations

- This product is made to use in dental restorations and should be used according to the manufacturer's instructions.
- Avoid using and mixing the material with other brands on the market.
- In case of contact with the oral mucosa, rinse with water and do not use for people who are in any way allergic to the components of this substance and if you have any allergic or skin sensitivities after using this substance, consult a specialist doctor.